


INSTRUCTIONS

Thank you for choosing our STI test!

To guarantee a reliable test result, it is important that you follow the instructions carefully. Please read this instruction manual before use, so you know exactly what to expect.

Contents of the package:

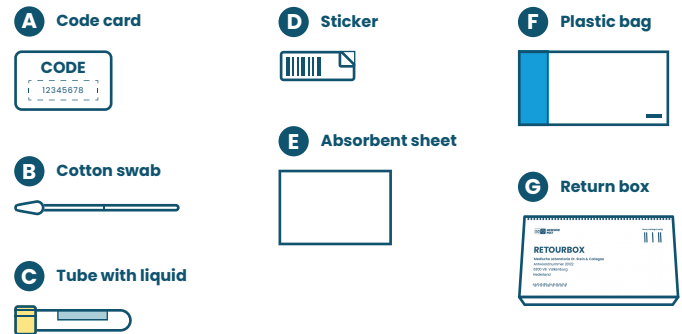

Instructions for taking the test



Take the code card **(A)** out of the return box and keep it. You will need the code later to request your results. Without the code, you will not receive your results.



Sit or stand in a way that feels comfortable for you, similar to when you insert a tampon. Take the cotton swab **(B)** and insert it 5 cm into the vagina. Then stroke along the vaginal wall in circular movements for 30 seconds.



Open the tube containing the liquid **(C)**. Insert the cotton swab and break it off at the break line. Then tighten the cap firmly and stick the sticker **(D)** straight onto the tube.



Place the tube with the absorbent sheet **(E)** in the plastic bag **(F)** and seal it tightly.



Put the plastic bag in the return box **(G)** and fold it closed. Use the adhesive strip on the side to easily seal the box.



You can put the return box in a PostNL letterbox. You can ask for your result after 24 to 36 hours from the moment your sample has arrived in our lab.

! Don't test until 14 days after unsafe sexual contact!